Grandi Maestri

Leonardo

ELETTROTHERAPIA PROGRAMMABILE

PROGRAMMABLE ELECTROTHERAPY

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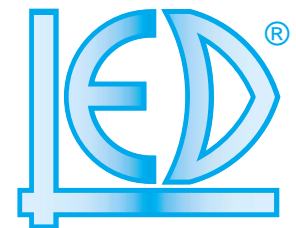
- GALVANIZATION
- IONTOPHORESIS
- MOTION THERAPY
- ANALGESIC THERAPY
- ANTI-CONTRACTURAL THERAPY
- ANTI-EDEMATOUS THERAPY
- MOTION STIMULATION THERAPY
- INHIBITORY THERAPY
- S.P.E.S. THERAPY

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LEONARDO delivers different types of currents suited for most of the therapy of painful syndromes or circulatory troubles, and for invigorating or hypertrophying treatment through passive gymnastics, and rehabilitation of normoinnervated and/or denervated muscular tissues.

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LEONARDO, realized with the more advanced electronic technologies, it is proposed in three models (Basic, Classic and Special). Each model is distinguished in other release on the base of the number of available channels (1 channel, 2 channels and 4 channels).

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LEONARDO has:

- **DIADYNAMIC CURRENT** assures the success of the treatment of painful inflammatory conditions of muscles, articulations, peripheral nerves and, in the sporting medicine, the rapid resolution of traumas and hematomas.
- **FARADIC CURRENT** which allows the stimulation of normoinnervated muscles in every treatments.
- **PULSE CURRENT**, that thanks to a constant increment of the intensity respect the time, can be used for the stimulation of partially innervated or denervated muscles.
- **DIRECT CURRENT**, that can be delivered singularly or mixed with diadynamic, faradic and pulse current, makes **LEONARDO** useful for the galvanization, the galvanic bath and the iontophoretic administration of drugs.
- **BIPOLAR CURRENT** (models Classic and Special) it allows the use for the invigorating and rehabilitative muscular stimulation. The symmetrical bipolarity of the stimulating waves improves the sensory acceptability of the stimulation, due to the soft perception of the delivered currents, avoiding other drawbacks to the electrolytic action of the monopolar stimulation.
- **INTERFERENTIAL CURRENT** (models Special) they assure the transfer of low frequency antalgic or motor stimulating current to the deep tissue levels, though the aid of higher frequencies that reduce the cutaneous impedance with consequent diminution of problems of irritation of the epidermis.

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The section Diagnosis in the models Special allows to carry out the evaluation of the figures of rheobase, cronaxie, accommodation and the strength/duration diagram.

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CODICE CODE	DESCRIZIONE ACCESSORI ACCESSORIES DESCRIPTION	LEONARDO							
		BASIC			CLASSIC			SPECIAL	
		1 CH	2CH	4CH	1 CH	2CH	4CH	2CH	4CH
00100.03	Cavo alimentazione 2MT 3x1mm SIEMENS-IEC <i>Power supply cable 2MT 3x1mm SIEMENS-IEC</i>	•/1	•/1	•/1	•/1	•/1	•/1	•/1	•/1
TROL003	Carrello dimensioni ripiano 40x35 cm <i>Trolley shelf size 40x35 cm</i>	◦	◦	◦	◦	◦	◦	◦	◦
00600.01	Elettrodo in gomma conduttiva 80 x 120 mm <i>Conductive rubber electrode 80 x 120 mm</i>	•/2	•/4	•/8	•/2	•/4	•/8	•/4	•/8
00600.02	Elettrodo in gomma conduttiva 50 x 50 mm <i>Conductive rubber electrode 50 x 50 mm</i>	•/2	•/4	•/8	•/2	•/4	•/8	•/4	•/8
00600.13	Elettrodo adesivo 45 x 80 mm (conf. 4 pz) <i>Pkg of 4 adhesive electrodes 45 x 80 mm</i>	•/1	•/2	•/4	•/1	•/2	•/4	•/2	•/4
00600.99NE	Adattatore Clip-femmina 4mm NERO <i>Clip-female adapter 4mm BLACK</i>	•/1	•/2	•/4	•/1	•/2	•/4	•/2	•/4
00600.99RO	Adattatore Clip-femmina 4mm ROSSO <i>Clip-female adapter 4mm RED</i>	•/1	•/2	•/4	•/1	•/2	•/4	•/2	•/4
00601.00	Busta spugna 80 x 120 mm <i>Envelope sponges 80 x 120 mm</i>	•/2	•/4	•/8	•/2	•/4	•/8	•/4	•/8
00601.01	Busta spugna 50 x 50 mm <i>Envelope sponges 50 x 50 mm</i>	•/2	•/4	•/8	•/2	•/4	•/8	•/4	•/8
00602.040	Fascia in velcro 8 x 40 cm <i>Fasting bandage size 8 x 40 cm</i>	•/1	•/2	•/2	•/1	•/2	•/2	•/2	•/2
00602.060	Fascia in velcro 8 x 60 cm <i>Fasting bandage size 8 x 60 cm</i>	•/1	•/2	•/2	•/1	•/2	•/2	•/2	•/2
00602.100	Fascia in velcro 8 x 100 cm <i>Fasting bandage size 8 x 100 cm</i>	•/1	•/1	•/2	•/1	•/1	•/2	•/1	•/2
2030NE-200	Cavo collegamento BAN/BANPROT 2m NERO <i>Cable connection 2mt BLACK</i>	•/1	•/2	•/4	•/1	•/2	•/4	•/2	•/4
2030RO-200	Cavo collegamento BAN/BANPROT 2m ROSSO <i>Cable connection 2mt RED</i>	•/1	•/2	•/4	•/1	•/2	•/4	•/2	•/4
BAG001	Borsa per trasporto Grandi Maestri <i>Grandi Maestri carrying bag</i>	◦	◦	◦	◦	◦	◦	◦	◦
LOCD001	Software gestione programmi Leonardo <i>Leonardo Software programs management</i>	◦	◦	◦	◦	◦	◦	◦	◦
LOKD001	Kit per funzione Diagnosi <i>Kit for diagnosis function</i>	—	—	—	—	—	—	◦	◦
LOTL001	Telecomando Leonardo <i>Leonardo Remote control</i>	◦	◦	◦	◦	◦	◦	◦	◦

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LED SpA